


star**PRO**tect-shield UP can be easily attached to your ZEISS or Swarovski Kepler optics.

1. If you have an adapter from star**Med** for the star**Light**, use the two long screws (A). If you are not using an adapter, take the two short screws (B).

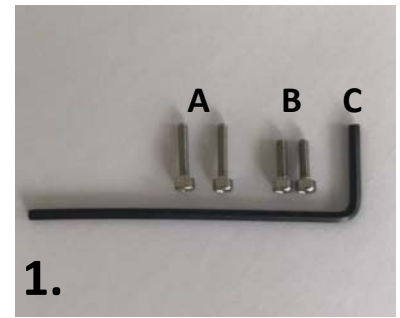

2. Both screws are stuck through the adapter (if present) and through the attachment. Fasten the screws with the Allen key © to the Kepler system until a firm hold is achieved (caution against the risk of over-tightening).



3. The inclination of the sign can be adjusted 0-90 ° independently of the supporter on the lower milled nut (D). This function is required to adjust the loupes. After optimally adjusting the prismatic loupes, the protection shield is reset to the desired inclination and fixed with the help of the two lock nuts (D).



4. Independently of the preset inclination of the protective face shield, the entire system can be hinged up and down as usual (flip-up).



Manufacturing and sales
star**Med** GmbH & Co. KG
Call +49 (0) 8092 - 850 447
Mail to: info@starmed-technik.de